

Quality controls

Thermo Scientific MAS QC Handling

Thermo Scientific™ MAS™ controls stored at -15 to -25°C

Prior to routine use



Thaw the vials **at room temperature (18-25°C)** on a **rocker** or with periodic gentle inversion until contents liquify.

1



IMMEDIATELY store at 2-8°C.

2



For controls with analytes that need a dropper tip, replace the vial cap with a dropper tip before the first use. **After the vial is empty, discard the dropper tip with the vial.**

3

Routine use



Before each use, **retrieve the control vial from the refrigerator and mix the contents** by gently inverting for several minutes.

1



Use the QC **IMMEDIATELY**. Open the vial and transfer the required quantity of control into a clean sample cup.

2



Replace cap immediately and store the opened vial at 2-8°C.

3

ONCE THAWED, DO NOT REFREEZE THE CONTROL.

For more information, refer to the Instruction for Use section in the package insert of your QC product.

Learn more at [thermofisher.com/MASproductivity](https://www.thermofisher.com/MASproductivity)

thermo scientific

Not all products are CE marked or have 510(k) clearance for sale in the U.S. Availability of products in each country depends on local regulatory marketing authorization status.

© 2022 Thermo Fisher Scientific Inc. All rights reserved. All trademarks are the property of Thermo Fisher Scientific and its subsidiaries unless otherwise specified. **CDD-FR-MTL-0314**