




Thermo Scientific
Parasitology testing solutions

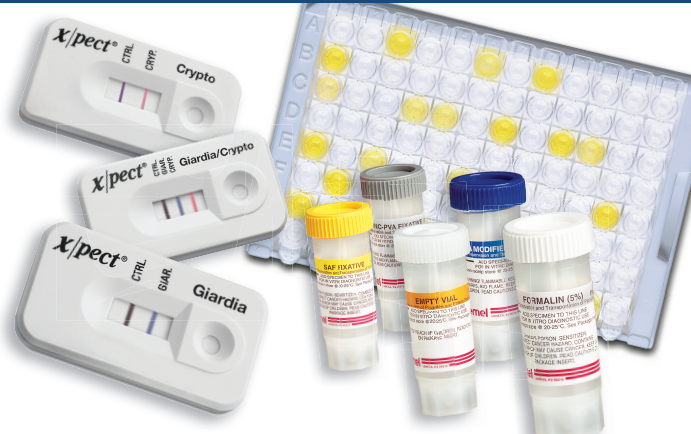

Quick, reliable

Giardia & Cryptosporidium testing



More solutions for parasitology testing

Depend on the industry's leading microbiology expert, Thermo Fisher Scientific, to find the right solutions to optimize your workflow for intestinal parasitic testing. Thermo Scientific products are designed with the highest degree of sensitivity and specificity to guide quality care. With timely and accurate results, you can effectively support patient well-being and satisfaction.



Xpect™

Rapid single or combination immunoassays for *Giardia* and *Cryptosporidium*



ProSpecT™

Microplate immunoassays for *Giardia*, *Cryptosporidium* and *E. histolytica* for high volume testing



Fecal Transport

Safe collection and preservation of fecal specimens, available in multiple pack sizes and sets



O&P Stains & Reagents

Detection and differentiation of intestinal parasites with high quality stains and reagents



“I need to drive patient satisfaction”

Make sample collection a one-time-only experience for your patients with **Remel Xpect** and **Remel ProSpecT**. Flexible fecal transport options, including Cary-Blair, reduce the possibility of sample incompatibility and the need for recollection. Additionally, making antigen testing routine allows for the collection of only one specimen compared to the two or three needed for O&P testing.

“I need rapid, crystal clear results”

With **Xpect**, you can obtain objective, color differentiated results for *Giardia* and *Cryptosporidium* in 15 minutes. For high volume testing, choose **ProSpecT** and get results in 100 minutes with true colorless negatives.



“I need to support consistent, quality care”

With proprietary technology from Remel, both **Xpect** and **ProSpecT** detect *Giardia* Specific Antigen (GSA 65), which can be found in fecal specimens even when visible evidence of trophozoites or cysts is lacking on O&P examination. Formation of cysts is extremely variable and may lag behind the onset of symptoms by a week or more. Other products detecting only the cyst antigen may miss infections in the early stages before the trophozoites transform into cysts.

“I need flexibility to work with challenging specimens”

Now you can get results from even the most difficult samples with **Xpect**. Whether you're dealing with a stool sample that is too viscous or mucoidal, **Xpect** provides you with a validated retest option.

“I need reliability & simplicity”

Thermo Scientific products include a full range of dependable and easy-to-use parasitology solutions for every step of your workflow. Whether you're looking for O&P stains, fecal transport options, ELISA or lateral flow rapid antigen assays, each product is supported by world-renowned microbiology and parasitology expertise.





Committed to patient satisfaction

Under the Thermo Scientific brand we deliver a range of solutions for the microbiology laboratory, supporting the delivery of fast, reliable detection. We understand that quality patient care is paramount and ensuring patient satisfaction is non-negotiable. With our commitment to bringing more to parasitology, you can confidently rely on our product quality and expertise to help you work faster and smarter.

Product Code	Product	Pack Size
R2450520	Xpect Giardia/Cryptosporidium	20 tests/kit
R2450020	Xpect Giardia	20 tests/kit
R2451020	Xpect Cryptosporidium	20 tests/kit
R2458496	ProSpecT Giardia/Cryptosporidium	96 wells/kit
R2458024	ProSpecT Giardia/Cryptosporidium	24 wells/kit
R2456096	ProSpecT Entamoeba histolytica	96 wells/kit
R21925	Cary-Blair w/ Indicator	15mL/vial; 120 vials/pack
R21945	Zinc PVA // Formalin 10%	2 vials/set; 120 sets/pack
R21922	Empty Vial	15mL/vial, 120 vials/pack
R21921	SAF Fixative	15mL/vial; 120 vials/pack
R21901	Fecal Concentrator Kit	36 tests/kit
R40035	Schaudinn's Fixative	250mL/bottle; each
R40217	Trichrome Stain Set	Kit
R40104	TB Kinyon Carbofuchsin	250mL/bottle; each
R40110	TB Methylene Blue	250mL/bottle; each
R40121	Methanol	250mL/bottle; each
R40124	Sulfuric Acid 1%	250mL/bottle; each
R40034	D'Antoni's Iodine	25mL/bottle; each

thermoscientific.com/microbiology

© 2013 Thermo Fisher Scientific Inc. All rights reserved.
All trademarks are the property of Thermo Fisher Scientific Inc., and its subsidiaries.

Contact Information:

International
+44 (0) 1256 841144
oxoid.info@thermofisher.com

USA
+1 800 225 6730
csemail@thermofisher.com

991-355

Thermo
SCIENTIFIC
Part of Thermo Fisher Scientific